


# BODIPACK +

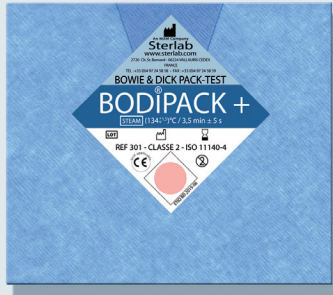

A ready for use  
Bowie & Dick  
test pack for steam  
sterilisation.  
EN 11140 - 4

## INSTRUCTIONS FOR USE

Check the pink spot indicator on the label to verify that the **BODIPACK +** has not already been used.

Place the **BODIPACK +** on the middle shelf of an empty autoclave with the label facing upwards.

Run a Bowie & Dick cycle for 3.5 min ( $\pm 5$  sec) at 134°C (0+1.5°C), in a cold or warm autoclave.

As soon as the cycle is finished, take the BODIPACK + test sheet out to check the results.

Do not forget to fill in the information on the test sheet for traceability.


Before use, store the **BODIPACK +** in a dry place at room temperature.



Sterlab

Année d'apposition : 1994



 Sterlab - 2720 Chemin St-Bernard - 06224 VALLAURIS CEDEX


Tel : +33 (0)4 97 24 58 58 - Fax : +33 (0)4 97 24 58 59

[www.sterlab.com](http://www.sterlab.com)

## INTERPRETATION OF RESULTS

### Fig. 1 - PASS

The stripes on the **BODIPACK +** test sheet are black or anthracite grey.  
The colour change is homogeneous.

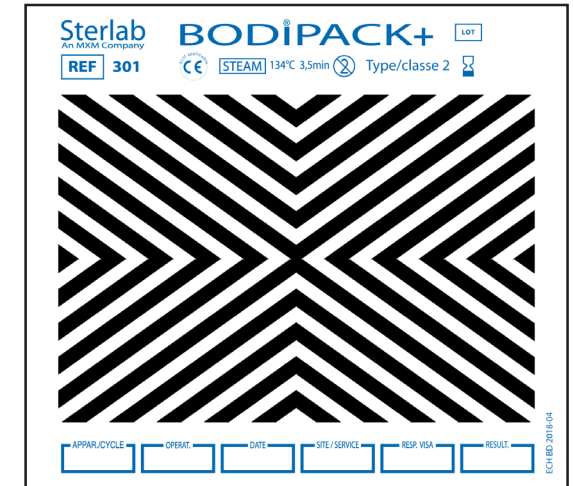

### Fig. 2 - FAIL

A decoloration appears in the centre of the **BODIPACK +** test sheet.  
It is not homogeneous and the colour change turns to brown.

