Fiche technique

EUROTEST®

Références 307-307D

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EUROTEST®

References 307-307D

EUROTEST® is a ready to use Pack Test of Bowie & Dick.

- Device for single use, dedicated to steam sterilisation
- To be run in a 134°C cycle (-0/+1,5°C) during 3,5 min (+5/-5") or 121°C cycle (-0/+1,5°C) during 15 min (+5/-5")
- Pack test dimensions: 95 x 95 x 10 mm
- Stability of the ink after exposure in compliance with the standard ISO 11140-1
- There are not known conditions or materials that can interfere with the correct intended use for the indicator's performances
- Latex, phtalate and lead free

Normative and regulatory references

- Type 2 chemical indicator, according to ISO 11140-1 norm
- Complies with the ISO 11140-4 Norm
- Complies with CE mark (if applicable according to countries) according to the European Regulation 2017/745 relative to medical devices - Medical device of class I

Pack content

- 1 test sheet with chemical reactive ink turning from green to black when exposed to steam and non reactive blue ink for legal mentions of traceability, according to ISO 11140-4 norm
- Filter cardboard sheets (grammage: 290 g/m²)
- Filter cardboard sheets (grammage: 290 g/m²) laminated with PET (thickness: 38 µm)
- 1 self-adhesive label

Extern Packing

- Packaging made of a self-adhesive label with traceability prints of legal mentions in compliance with the ISO 11140-1 Norm
- Type 1 steam indicator (dot) according to ISO 11140-1 norm, turning from pink to brown/black
- Directions for use (OK / STOP) given on the extern packaging
- Location booked for the user to register the autoclave number and the date

Transport Packaging

- Packaged in white wavy cardboard boxes by 50 units, overpacked in Kraft Havana cardboards by 200 units (4 x 50) for reference 307 or 1 box of 20 units for reference 307D.
- Packs protected by a plastic film (thickness: 12,5 µm) and air bag wedges
- Packagings in compliance with the Packaging Directive 94/92/EC of 20/12/1994 and to the french decree 2007-1467 of 12/10/2007 concerning the requirements for the environment

Storage Conditions

- Store in a dry place (30%/60% HR), at room temperature (+5°C/+35°C)
- Shelf-life of 3 years